



GC Fujirock® EP Classic
GC Fujirock® EP Premium
GC Base Stone

GC's family
of **stones**

Your
aesthetics
start here

GC

Superior quality is what makes the GC family of hard-die stones among the most popular in Europe. Used in the production of all kinds of dental prosthetics, GC stones are specifically designed to produce consistently accurate models and dies, fully meeting the requirements of the modern dental laboratory.

True to the systematic approach of GC - "One proven stone family for all requirements and applications" - we present three stones that can be combined perfectly to create the foundation of your aesthetics:


GC Fujirock® EP **Classic**, a truly all-round stone

GC Fujirock® EP **Premium**, an improved stone for high-performance techniques

GC Base Stone, an exceptionally free-flowing stone for bases with minimal expansion

Each stone comes in a range of colours that allow you to create shade combinations that bring out the best in your work ... and your laboratory.

GC's family of stones ... the foundation for success.

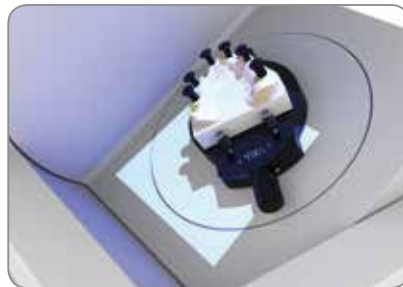



Three stones, unlin

GC Fujirock® EP Classic

GC Fujirock® EP Classic lives up to its name. One of the world's most popular Type 4 dental die stones, it is "the classic". It is ideal for all kinds of prosthetics, combining high precision, outstanding edge hardness and high-pressure stability. GC Fujirock® EP Classic is very fluid yet still thixotropic for excellent handling properties and easy pouring with no air bubbles.

Choose the colour that best suits your daily work:

- Golden Brown
- Topaz Beige



GC Base Stone

GC Base Stone is ideal for basing dentate arches, with either GC Fujirock® EP Classic or Premium, the stones makes them the optimal combination for c

Choose your own complementing colour:

- Terracotta Red
- Royal Blue



Unlimited possibilities



GC Fujirock® EP Premium

GC Fujirock® EP Premium has been specially developed for high performance and sensitive technologies such as extended implant cases and complex full-ceramic indications.

The improved powder composition of GC Fujirock® EP Premium and refined colour setting make this special stone the perfect choice for a variety of applications.

Choose the colour that best suits your refined works:

- Pastel Yellow
- Polar White
- Inca Brown
- Titanium Grey



opposing models and study models. When used
the single "GC quality" standard behind the
creating superb master models quickly and easily.



The Art of presenting Aesthetics



Fujirock® EP Classic & Premium

Fujirock® EP – Classic

- 890222 Golden Brown 5 Kg pack
- 890226 Golden Brown 12 Kg pack
- 890225 Topaz Beige 5 Kg pack



GC Fujirock® EP Classic	
Golden Brown / Topaz Beige	
P/L Ratio	20 ml/100 g
Manual mixing	15 sec.
Vacuum mixing	45 sec.
Setting time	10-13 min.
Removal time	30-40 min.
Setting expansion (after 2 h)	≤ 0.09 %
Compressive strength	≈ 53 Mpa

Fujirock® EP – Premium

- 890364 Pastel Yellow 4 Kg pack
- 890365 Pastel Yellow 11 Kg pack
- 890366 Polar White 4 Kg pack
- 890367 Inca Brown 4 Kg pack
- 890368 Titanium Grey 4 Kg pack



GC Fujirock® EP Premium	
Pastel Yellow / Polar White / Inca Brown / Titanium Grey	
P/L Ratio	20 ml/100 g
Manual mixing	15 sec.
Vacuum mixing	45 sec.
Setting time	9-12 min.
Removal time	25-35 min.
Setting expansion (after 2 h)	≤ 0.09 %
Compressive strength	≈ 60 Mpa

Base Stone

Base Stone

- 890228 Terracotta Red 12 Kg pack
- 890229 Royal Blue 12 Kg pack



Base Stone		
Terracotta Red / Royal Blue		
	Flow	Thixo
P/L Ratio	26 ml/100 g	21 ml/100 g
Manual mixing	15 sec.	15 sec.
Vacuum mixing	45 sec.	45 sec.
Setting time	9 min.	7,5 min.
Removal time	30-40 min.	30-40 min.
Setting expansion (after 2 h)	≤ 0.08 %	≤ 0.08 %
Compressive strength	≈ 45 Mpa	≈ 50 Mpa

All packs with easy open / easy close plastic lid. Hermetic seal protects against humidity.

GC EUROPE N.V.
 Head Office
 Researchpark
 Haasrode-Leuven 1240
 Interleuvenlaan 33
 B-3001 Leuven
 Tel. +32.16.74.10.00
 Fax. +32.16.40.48.32
 info@gceurope.com
 http://www.gceurope.com

GC UNITED KINGDOM Ltd.
 Coopers Court
 Newport Pagnell
 UK-Bucks. MK16 8JS
 Tel. +44.1908.218.999
 Fax. +44.1908.218.900
 info@uk.gceurope.com
 http://uk.gceurope.com

